

3diag - C5 - 800

Complement C5 - for *IMMAGE*[®] 800

REF TD-42561

IVD CE

ANNEX to Instruction for Use:

Application for *IMMAGE*[®] and *IMMAGE*[®] 800 Immunochemical Systems

PROTOCOL INFORMATION

Reagent Lot Number:	(^{*1})	Units:	mg/dL or mg/L
Cartridge Lot Number:	(^{*1})	Protocol:	Non-competitive Nephelometry
Cartridge Serial Number:	(^{*1})	AGXS Limit:	not defined
Reagent Expiry date:	(^{*1})		
Sample or Dilution Volume:	40 µL	Diluent:	DIL1 (^{*2})
Reaction Buffer Volume:	195 µL	Buffer:	BUF1 (^{*3})
Calibration Dilution:	1 : 5	Gain:	3
Compartment A Volume:	30 µL	Sample Dilution:	1 : 5
Compartment B Volume:	8 µL	Reaction Time:	3.0 minutes

CALIBRATION INFORMATION

Levels:	6	Replicates:	2
Update Level:	4 (^{*4})	Replicates:	2 (^{*4})
Model:	Logistic 4 parameters		

NOTES

- (^{*1}): See the labels of the Reagent Cartridge.
 (^{*2}): Beckman Coulter P/N no. 447640. For the *IMMAGE*[®] it has to be defined as DIL10.
 (^{*3}): Beckman Coulter P/N no. 447650. For the *IMMAGE*[®] it has to be defined as BUF10.
 (^{*4}): The use of the calibration update, with only one calibrator level, is discouraged.