

3diag - SAA - 800

Serum Amyloid A - for *IMMAGE*[®] 800

REF TD-42881

IVD CE

IFU Annex: *IMMAGE*[®] 800 Immunochemical Systems
 Application Proposal (Serum)

PROTOCOL INFORMATION

Reagent Lot Number:	(*)	Units:	mg/L
Cartridge Lot Number:	(*)	Protocol:	Non-competitive Nephelometry
Cartridge Serial Number:	(*)	AGXS Limit:	not defined
Reagent Expiry date:	(*)		
Sample or Dilution Volume:	5 µL	Diluent:	DIL1 (*)
Reaction Buffer Volume:	200 µL	Buffer:	BUF 800-A (*)
Calibration Dilution:	1 : 1	Gain:	1
Compartment A Volume:	16 µL	Sample Dilution:	1 : 1
Compartment B Volume:	20 µL	Reaction Time:	5.0 minutes

CALIBRATION INFORMATION

Levels:	6	Replicates:	2
Update Level:	4 (*)	Replicates:	2 (*)
Model:	Logistic 4 parameters		

NOTES

- (*1): See the labels of the Reagent Cartridge.
- (*2): Beckman Coulter P/N no. 447640.
- (*3): For the *IMMAGE*[®] 800 it has to be defined as BUF10 to BUF15.
- (*4): The use of the calibration update, with only one calibrator level, is discouraged.

Modifications highlighted in blue | .